Development and Application of An Immunohistochemistry-based Clinical Assay for Evaluating Folate Receptor Alpha (FRα) Expression in the Clinical Setting

Abstract 3400A

Jianhua Zhao¹, Alyssa LaBelle¹, Olga Ab¹, Krista Acosta², Al Yates², Yinghui Zhou¹, Angela Romanelli¹ ImmunoGen, Inc., Waltham MA (1) and Ventana Medical Systems, Inc., Tucson, AZ (2)

INTRODUCTION


A biomarker-based patient selection strategy, coupled with the co-development of a companion diagnostic, is key to the successful development of molecularly targeted cancer therapeutics.


FR α is a glycosyl-phosphatidylinositol-linked membrane protein. Expression of FR α is rare in normal tissue, but frequently elevated in several solid tumor types including epithelial ovarian cancer (EOC), endometrial cancer and lung adenocarcinoma. Mirvetuximab soravtansine (IMGN853) is an antibody-drug conjugate (ADC) consisting of an anti-FR α antibody linked to DM4, a highly cytotoxic maytansinoid, *via* a cleavable disulfide linker. Preliminary evidence of clinical activity has been observed in patients receiving mirvetuximab soravtansine treatment in the first-in-human phase I clinical trial.

In this study, we report the development of an IHC-based FR α clinical-trial assay to support patient selection for mirvetuximab soravtansine. To this end, an IHC compatible murine monoclonal antibody with high specificity for human FR α was generated (Clone FOLR1-2.1). The antibody was used to develop and optimize an IHC assay that covers a broad dynamic range of FR α expression, allowing discrimination among high, moderate and low levels of expression. This semi-quantitative assay has been analytically validated per CAP/CLIA guidelines and supports intended indications including ovarian cancer, endometrial cancer and NSCLC. Additional data generated using this assay demonstrated variation in levels of FR α expression between ovarian and endometrial cancer, as well as among different histological subtypes of endometrial cancer. Comparison between pairs of primary and metastatic samples from the same patients was also carried out to understand the utility as well as potential limitation of this assay. These data help us to refine the clinical development strategy as well as the strategy for patient selection.


Generation of FOLR1-2.1: a FRα specific antibody for IHC


- Murine monoclonal antibody FOLR1-2.1 generated using standard hybridoma technology
- The immunogen: recombinant Fc-FRα
- Primary screening: FACS with fixed/denatured FRα+ cells

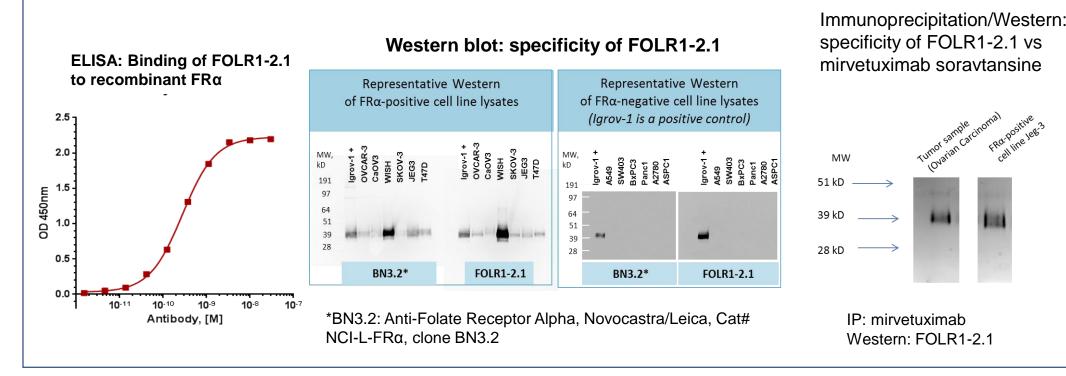

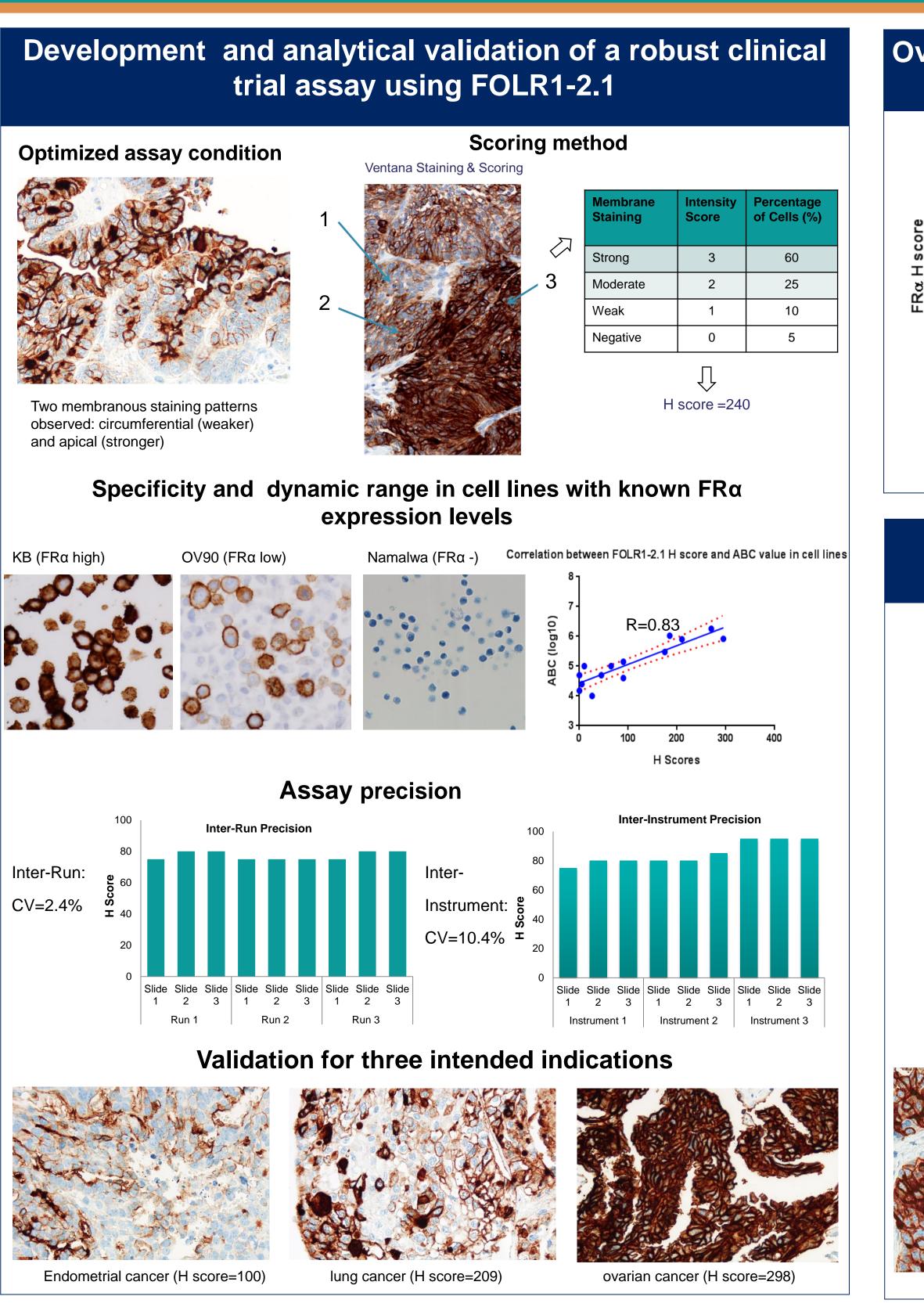


Ovarian cancer more likely to show high FRα expression than endometrial cancer FRa expression in different tumor types FRa expression in different tumor types (primary or metastasis) ••• ຍັ 200 -201-300 ■101-200 •••• 1-100 20 22 100 -Greater proportion of ovarian cancer (OVC) showed high FRa expression than endometrial cancer (EMC) Pair1 Serous ovarian Endometrial carcinoma with serous histology more carcinoma likely to show high FRα expression FRα expression in histological subtypes Pair 2: of endometrial cancer **Papillary serous** endometrial carcinoma • -epitopes. NOS: Not Otherwise Specified Serous and endometrioid Endometrioid carcinoma Serous carcinoma mixed carcinoma (H score=50) (H score=280)(H score=150)

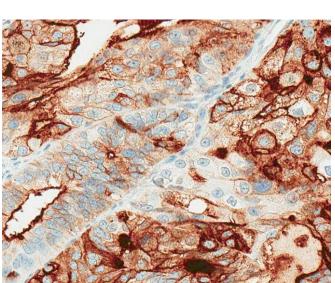

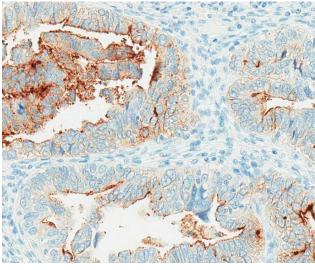

AACR annual meeting, April 18-22, 2015 ©2015 ImmunoGen, Inc., Waltham MA, USA

IMMUNOGEN

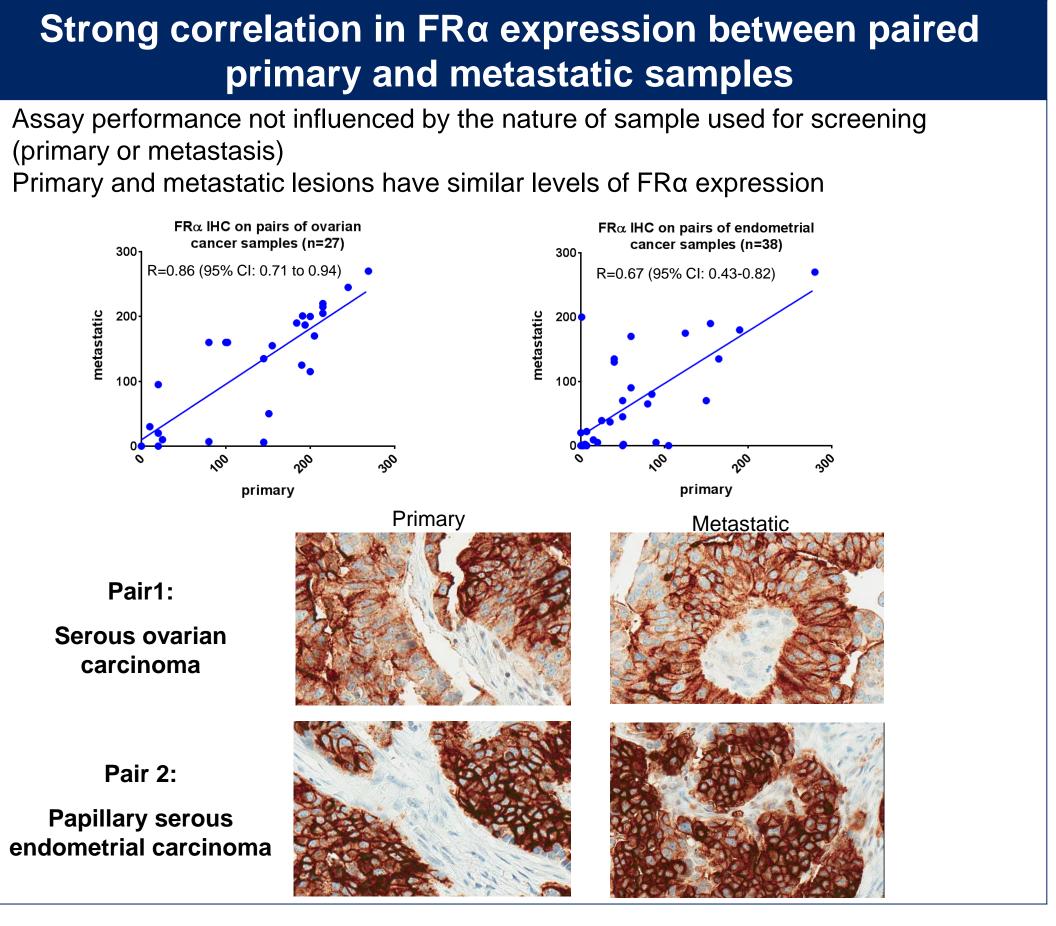

SUMMARY

An immunohistochemistry-compatible FR α -specific antibody was generated and selected from a panel of monoclonal antibodies for its ability to bind to denatured

A robust IHC assay for detecting FRα expression in archival tumor tissue was developed and analytically validated, supporting patient selection in clinical trials. Staining of tumor samples using this assay demonstrated that a greater proportion of ovarian samples exhibited high FRα expression than endometrial samples.

Among endometrial samples, a greater proportion of tumors with the serous histology exhibited high FR α expression than the other histological subtypes.

Comparison of paired primary/metastatic lesions from the same patients showed a strong correlation of FRα expression levels between the two, demonstrating consistency and utility of the assay irrespective of the sample source.

6. This assay is currently in use for patient selection in clinical trials for mirvetuximab soravtansine.

